

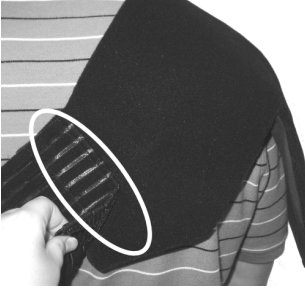




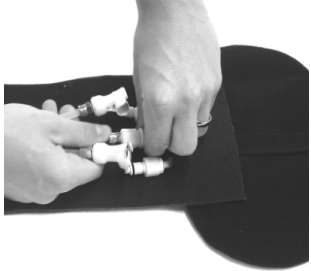
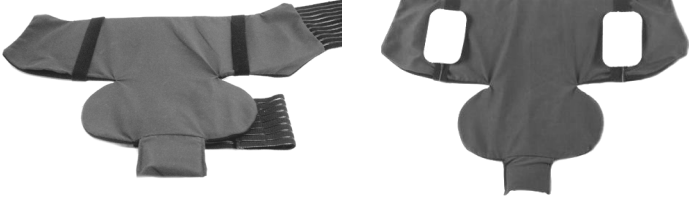


PATIENT INSTRUCTIONS

Kinex SO Shoulder

 <p>Insert bladder into exchanger and secure with hooks & loop closure</p>	 <p>Position the therapy wrap over the shoulder as shown</p>	 <p>Attach shoulder strap as shown</p>	 <p>Wrap underneath opposite arm and secure second end of hook & loop strap</p>
 <p>Attach bicep strap as shown</p>	 <p>Wrap underneath bicep and secure second end of strap</p>	 <p>Attach arm sling by connecting to the bottom of the wrap</p>	 <p>Connect the umbilical hose to the bladder. The connectors will make a "click" sound, indicating a secure connection.</p>
<div style="display: flex; justify-content: space-around; align-items: center;">  </div> <p>OPTIONAL: <i>TENS Garment</i> Conductive garment can be placed anywhere on the exchanger.</p>			

Note: Loosen wrap if water circulation is compromised.

Cleaning Instructions: Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

For Single Patient Use Only.

Distributed by:

KINEX
Phase I Rehab